




INSTRUCTIONS FOR USE IN THE CASE OF CHOKING

If a casualty is a victim of choking

- 1 Encourage the victim to cough.
- 2 If the cough is not effective give 5 back slaps.
- 3 If obstruction has not been cleared, follow by 5 abdominal thrusts.

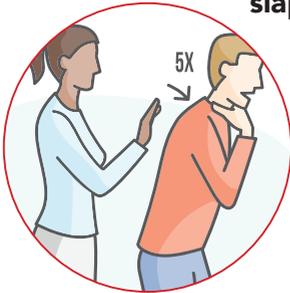

Repeat sequence until blockage is removed or victim becomes unconscious

4




Once unconscious or if steps 2 and 3 cannot be applied, call emergency services and use LifeVac.

Place over mouth and nose creating a good seal with correct sized mask

Push in handle/bellows

Pull handle up sharply

1 PLACE 2 PUSH 3 PULL

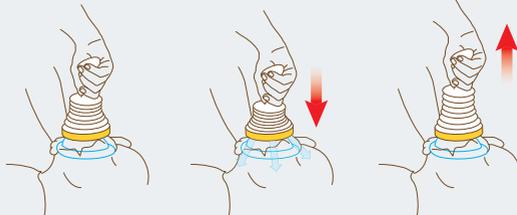

REPEAT UNTIL BLOCKAGE IS REMOVED OR UP TO 5 TIMES

If blockage is not removed commence CPR

LifeVac Europe LTD Unit 2,
Hill Village Business Park, Barnstaple, EX320RP

www.lifevac.uk

Telephone 01271 316044

Rev.3. Date 11.2.19



LIFEVAC EUROPE LTD